EURO Glider

CE 0120

NEDERLANDS

KWALITEIT

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ENGLISH

QUALITY

EUROGLIDER-condoms are made of natural rubber latex. Immediately after production each EUROGLIDER-condom is individually electronically tested for leakage. EUROGLIDER-condoms conform to the European quality standard EN ISO 4074:2002.

CONDOMS AND CONTRACEPTION

EUROGLIDER-condoms, when used properly, are highly effective against pregnancy. Therefore, please read the instructions below carefully.

CONDOMS AND AIDS

EUROGLIDER-condoms, when used properly, may help reduce the risk of catching or spreading AIDS and other Sexually Transmitted Diseases.

CONDOMS AND RELIABILITY

No condom can guarantee 100% effectiveness. Even a top quality condom like EUROGLIDER, **can experience a pinhole, usually always caused by mishandling during unpacking or putting on.** Almost all accidents with condoms are caused by improper use. So for maximum benefits it is important to read and follow-up the instructions below on **how to use a condom.**

HOW TO USE A CONDOM PROPERLY?

1. Open the condom-foil only right before use.
2. Open the condom-foil by tearing the foil in the middle. **Do not use sharp objects.**

3. Do not roll off or inflate the condom beforehand.
4. If you have a foreskin, pull it back before rolling on the condom.
5. Hold the nipple end between thumb and index to remove any remaining air in the tip of the condom.
6. Avoid all contact between penis and vagina, anus or mouth before the condom is put on.
7. Place the condom on the top of the erect penis and slowly unroll to the basis of the penis.
8. Beware not to damage the condom with sharp or long fingernails or rough skin.
9. If the condom does not unroll easily it has been put on incorrectly. Discard it and use a new condom.
10. After ejaculation, carefully withdraw the penis while it is still semi-erect. Hold onto the rim of the condom so that the condom does not slip off or spill semen.

Always remember the following rules:

1. **Never use oil-based lubricants**, such as those made with a petroleum jelly, mineral oil, vegetable oil or cold cream. These will dramatically reduce the strength of the condom immediately. **Only use water-based lubricants** if extra lubricant is desired. Safe Lubricants are KY, For Play, Wet Stuff or Eros.
2. Do not reuse a condom.
3. Store condoms at room temperature, avoid excessive heat and do not expose them to direct sunlight.
4. If your partner is using a medication that is applied to the genital areas, please consult a doctor to ensure that it will not damage the condom.
5. Euroglider-condoms are made of natural rubber latex, which may cause allergic reactions in some individuals. When you get an irritation after you used a condom, consult a doctor.
6. Never throw a used condom in a toilet.

EXPIRATION DATE

For safety and quality assurance, the expiration date is printed on each condom pack and on each individual foil of the EUROGLIDER-condom.

INFORMATION

Should you require any further information about condoms, their usage, contraception or sexually transmitted diseases, please contact your doctor, local Family Planning Association, General Practitioner or Sexual Health Clinic.

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