DENMARK

DK

SCANDANAVIA

S

NORWAY

N

HVAD ER DET?

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GB

HOW DOES FEMIDOM WORK?

- FEMIDOM works like a male condom, but instead of covering the penis it lines your vagina.
- When used according to the instructions FEMIDOM protects against pregnancy, and provides a barrier to sexually transmitted infections, including HIV.
- Like a male condom, it may take a couple of goes to get used to using it.
- Each FEMIDOM must be used only once.

INSTRUCTIONS FOR USE

- 1 Tear down from arrow on sachet, and pull out the condom.
- 2 FEMIDOM should be inserted before any genital contact takes place. The small ring which lies within the closed end of the sheath helps you insert FEMIDOM much like a tampon and holds the sheath in place in the vagina. To insert, firmly hold the small ring between your thumb and middle finger.
- 3 Find a comfortable position; you may like to lie down, sit with your knees apart or stand and place one foot up on a chair. Insert the squeezed ring into the vagina pushing it inside as far as possible. You will find that putting your index finger between your thumb and middle finger will help you to guide FEMIDOM into place.
- 4 Put finger inside the sheath and push the small ring up as far as it can go, like a tampon. Some women may find that insertion is fully completed by the penis when it enters the vagina.
- 5 It is normal for part of the sheath to hang outside your body. The outer ring helps keep the sheath in place and will be flat against you when the man is inside you.
- 6 Make sure you guide your partner's penis inside the sheath. Don't be concerned if FEMIDOM moves a little. As long as your partner's penis remains inside the sheath and the outer ring remains outside your body, then FEMIDOM is working.
- 7 To remove, twist the outer ring and gently pull. FEMIDOM must be removed before risk of spilling any sperm, immediately after loss of erection. Do not dispose of FEMIDOM down the toilet.

LUBRICATION

FEMIDOM is pre-lubricated to provide comfort. However, every individual has varying degrees of natural lubrication and some couples have found that FEMIDOM provides too much lubrication in which case any excess can be wiped off with a tissue. Other couples feel there is too little, in which case any brand of additional lubricant can be used with FEMIDOM. Should you find that the sheath makes a rustling noise or that your partner is pushing the inner ring inside you, you may find it helps to put extra lubrication inside the sheath or on the penis. Because of the design of FEMIDOM there is no need to use spermicide, however since FEMIDOM is made of polyurethane, it is safe to use any extra lubricant or spermicide you like.

REMEMBER

A drop of semen may contain millions of sperm and could cause a pregnancy or carry an infection. If you think there is any chance that you have been accidentally exposed to semen or that sperm may have entered your vagina, (eg if the outer ring has been pushed into your vagina) see your doctor or clinic as soon as is convenient, and certainly within 3 days, to ask about emergency contraception. If you are worried about infection, get medical advice.

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